


Project Type Instructions	Date: January 7 2021 Customer: Thermor Ltd Job Number: TH5821 Item Number: SC423 Designer: Alex Vranjesvic	Revision: A  Contact: Graphic's Department graphics@thermor-ins.com
UPC (For Reference Only)  0 57475 93423 1	Reason For Project: New Item: New Branding: Design Update: X N/A:	Artwork is not to be amended or scaled. If any changes are required please send through Thermor office with instructions, and we will amend here and resend artwork through. PLEASE RETURN WITH SIGNATURE 

BIOS living

Jumbo Digital Display Bath Scale

Instruction Manual

SC423

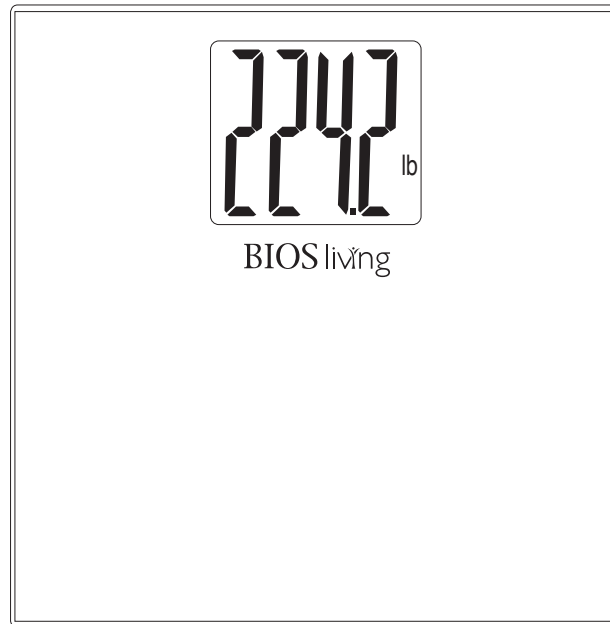

Introduction

Thank you for purchasing the BIOS Living Jumbo Digital Display Bath Scale. This scale was developed with state-of-the-art electronics. This precision device has a weight capacity of 550 lbs / 250 kg and is equipped with an extra large blue backlit screen.

Features

- Auto ON / OFF
- Auto zero reset
- Large blue backlit LCD (4" x 3½" / 10.16 x 8.89 cm)
- LB/KG/ST switchable
- Low battery / over-load indication
- Weight capacity: 550 lbs / 250 kg

Battery Installation (3 x AAA batteries included)

Open the compartment on the back of the scale and insert the batteries according to their polarity. Please be aware that inserting the batteries incorrectly may cause permanent damage.

NOTE: If you do not intend to use this scale for a long period of time, it is advisable to remove the batteries before storing it away.

Using Your Scale


- Select your preferred measurement by turning the scale over and pressing the button on the back to switch between kg or lb.
- Place scale on a hard flat surface

Unit





NOTE: If the scale is placed on a soft surface, such as carpet or an uneven floor, the results may be inaccurate or an error message may occur.

- Step onto the scale gently, placing your feet evenly apart, and stand still until the reading blinks 3 times.

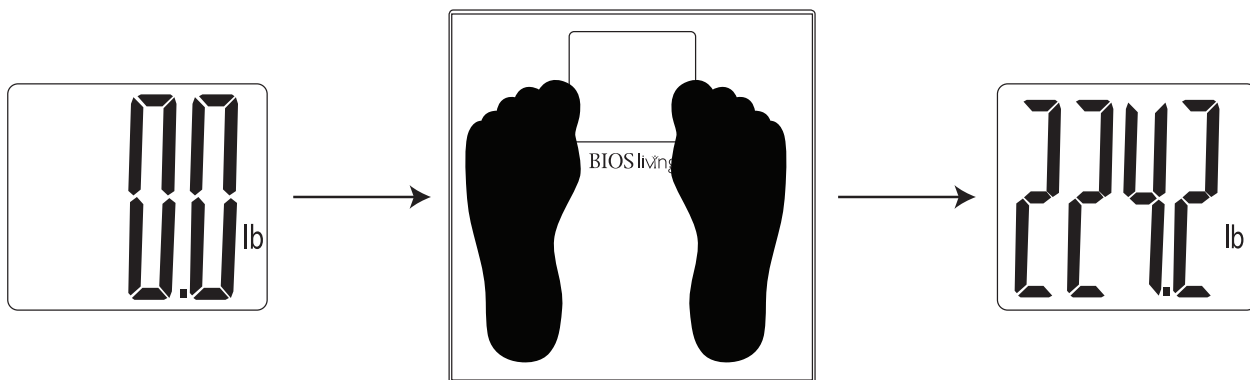

- The scale will shut off automatically after 15 seconds or once weight is removed.

Troubleshooting

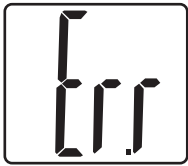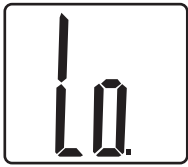Error	Problem	Solution
	An error has occurred when taking a measurement.	Step off the scale and wait a few minutes before trying again. Note: Do not use if you exceed the maximum weight capacity.
	Battery power is running low.	Remove batteries, wait 10 seconds and reinsert them into the scale. If error occurs again, replace the batteries.

Care and Maintenance

- Always keep the scale in a horizontal position and do not put in wet or hot conditions
- Avoid hitting or dropping the scale
- Do not overload the scale, as this will damage the sensor
- Do not immerse this product in water
- Clean the scale with a soft, damp cloth. Do not use chemicals or harsh abrasives
- Keep the scale surface dry and do not use on a slippery floor

Five Year Warranty

The Jumbo Digital Display Bath Scale has a 5 year warranty to be free of manufacturing defects in materials and workmanship under normal applications for 5 years of the original owner. If this product becomes inoperable due to defect and requires repair, return the product with all component pieces and proof of purchase to the address listed below. This warranty does not cover any shipping/transport costs. This warranty does not apply if the product is subject to misuse, neglect, rough handling or damage.

Ship the unit prepaid and insured (at owners option) to:

Thermor Ltd.

Attn: Repair Department
16975 Leslie Street
Newmarket, ON L3Y 9A1

www.biosmedical.com

Email: support@biosmedical.com