

Premium Analog Scale - Instruction Manual SC404

Introduction

Thank you for purchasing the BIOS Living Premium Analog Scale.

This scale is equipped with an easy to read dial and has a capacity of 330 lbs / 150 kg.

Features

- Easy to read 7" / 17.7 cm dial
- Solid steel construction
- Easy to clean non-slip platform
- Accurate to 330 lbs / 150 kg

Operation

1. Place the scale on a hard, flat surface (avoid carpet or other soft surfaces).

NOTE: If the scale is placed on a soft surface, such as a carpet, the results may be inaccurate.

2. Step onto the scale slowly and gently (do not jump onto the scale). Stand vertically without swaying and place your feet evenly apart - the dial will display your weight.

3. **Recalibration:**

Adjust the pointer on the dial with your finger (as shown in **Figure A**) to '0' for recalibration.

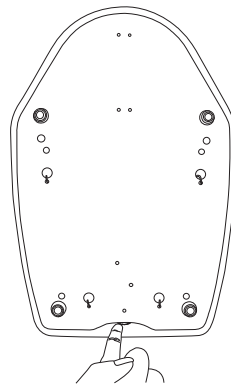


Figure A

Advice for Use and Care

- Avoid using device when wet - platform will be slippery when wet. Keep scale dry.
- Avoid excessive impact or vibration to the unit.
- Clean the scale with a damp cloth and avoid water getting inside scale.
- Store scale in horizontal position.

Five Year Warranty

The Premium Analog Scale has a 5 year warranty to be free of manufacturing defects in materials and workmanship under normal applications for 5 year of the original owner. If this product becomes inoperable due to defect, return the product with all component pieces and proof of purchase to the address listed below. This warranty does not cover any shipping/transport costs. This warranty does not apply if the product is subject to misuse, neglect, rough handling or damage.

Ship the unit prepaid and insured (at owners option) to:

Thermor Ltd.

Attn: Returns Department
16975 Leslie Street
Newmarket, ON L3Y 9A1

www.biosmedical.com

Email: support@biosmedical.com



BIOS | Medical
MANUFACTURED BY / FABRIQUÉ PAR :
THERMOR LTD.
16975 LESLIE STREET
NEWMARKET, ON L3Y 9A1
MADE IN CHINA / FABRIQUÉ EN CHINE
WWW.BIOSMEDICAL.COM

TH6774