BIOS living

Retro Mechanical Scale

Instruction Manual

SC402

Introduction

Thank you for purchasing the BIOS Living Retro Mechanical Scale. This scale is equipped with an easy to read dial and has a weight capacity of 280 lbs / 127 kg.

Operation

1. Place the scale on a hard, flat surface. Always weigh yourself on the same scale placed on the same surface for consistency.

NOTE: If the scale is placed on a soft surface, such as a carpet or on an uneven floor; the results may be inaccurate.

2. Step onto the scale slowly and gently (**do not** jump onto the scale). Stand vertically without swaying and place your feet evenly apart - the dial will display your weight.

3. Recalibration:

Adjust the pointer on the dial with your finger (as shown in Figure A) to '0' for recalibration.

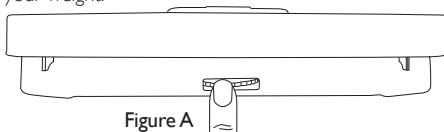
Figure A

Advice for Use and Care

- Avoid using device when wet - platform will be slippery when wet. Keep scale dry.
- Avoid excessive impact or vibration to the unit.
- Clean the scale with a damp cloth and avoid water getting inside scale.
- Store scale in horizontal position.

Five Year Warranty

The Retro Mechanical Scale has a 5 year warranty to be free of manufacturing defects in materials and workmanship under normal applications for 5 years of the original owner. If this product becomes inoperable due to defect and requires repair; return the product with all component pieces and proof of purchase to the address listed below. This warranty does not cover any shipping/transport costs. This warranty does not apply if the product is subject to misuse, neglect, rough handling or damage.

Ship the unit prepaid and insured (at owners option) to:

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Thermor Ltd.
THERMOR LTD.